

PLIWA Löffelreiniger

Concentrate for preparation of a working solution for cleaning of dental impression trays



Product description

Concentrate for preparation of a working solution for cleaning of dental impression trays.

Area of application

Suitable for cleaning of dental impression trays of stainless steel and plastic.

Special advice

Read safety datasheet before use.

Close container immediately after each use.

The product does not have any corrosive or decomposition effects that remove material residues.

Shelf life and expiry date

Shelf life of working solution: 1 day.

Use concentrate within 12 months after opening.

See product label for expiry date. Do not use the product after the expiry date.

Storage advice

Storage between 5°C and 25°C. Keep container tightly closed in a well-ventilated place. Protect from direct sunlight. Keep container in an upright position. Keep away from ignition sources. Keep away from food, drink and animal feedingstuffs. Only store in original container. Keep locked up and out of the reach of children.

Ordering information

Single package	Packaging unit	REF
1 L cylinder bottle	10 x 1 L (box)	10-200
5 L canister		10-203

Dosing aids for canisters

Dosing pump	50006
Drainage tap	50009

Application

Put on protective clothing / gloves and, if necessary, eye protection before cleaning.

1) Prepare working solution. 5-10% depending on the degree of soiling. Example for a 10% working solution: Mix 900 ml drinking water (max. 30°C) + 100 ml concentrate.

2) Clean dental impression trays roughly from attaching material (alginates, cement, silicone) and soak them in the working solution if possible over night.

3) Remove the dental impression trays from the working solution and remove residues of material with a brush.

4) Rinse dental impression trays under running water.

Labelling according to regulation (EC) No. 1272/2008 (CLP)



For information on labelling according to regulation (EC) No 1272/2008 (CLP) see safety datasheet and label.

Labelling according to regulation 648/2004/EC on detergents

≥5% - <15% non-ionic surfactants
< 5% anionic surfactants
disinfectants, preservations agents
(METHYLISOTHIAZOLINONE, BENZISOTHIAZOLINONE)

Further information

For further information on the product or individual questions contact us by e-mail (info@pliwa.de) or telephone (+49 (0) 5661-7317-0).



PLIWA Hygiene GmbH

Pliwastraße 2
34323 Malsfeld
www.pliwa.de

Tel. 05661 / 73 17 0
Fax 05661 / 73 17 10
info@pliwa.de

PLIWA Löffelreiniger

Concentrate for preparation of a working solution for cleaning of dental impression trays

Datasheet according to regulation (EG) 648/2004 Annex VII Section D

Name of product: PLIWA Löffelreiniger

Manufacturer: PLIWA Hygiene GmbH, Pliwastr. 2, 34323 Malsfeld, Germany

Ingredients:

AQUA
ISOPROPYL ALCOHOL
1-Heptanol, 2-propyl-,7EO
PROPYLENE GLYCOL
SODIUM COCETH-30 SULFATE
GLYCOLIC ACID
BENZOTRIAZOLE

Link to Pharmacos-Website: http://ec.europa.eu/consumers/sectors/cosmetics/cosing/index_de.htm

